

Implant Remover

FIXTURE REMOVER KIT SD-FR

For failed implants or implant marginal bone loss by excessive torque, remove the implant fixture by Fixture remover in safety.



Fixture remover screw

- Fixed in and remove implant fixture by turning counterclockwise.
- * In case that Fixture remover screw cannot be separated from the fixture because of excessive torque, forceps or mini vise can be used to grab and loosen the fixture by turning clockwise.



For Fixture remover screw

Turning handle


- Handle tool with Fixture remover screw for manual use.

TH-03



Wrench – Long

- Used with Fixture remover screw manually.

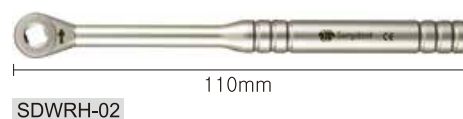

Ratchet extension

- Connected with wrench when the surgical site is not accessible.

RE-02



Handpiece condenser

- Used to connect Fixture remover screw with implant engine.

SDADP-01

